

# Hegenberger Retractor®

A1390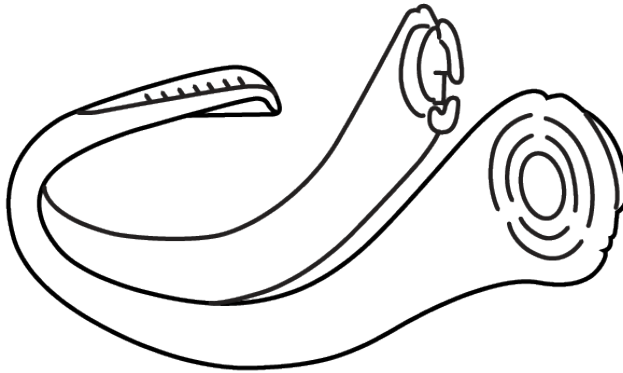

sv en no da fi de fr nl pl et lv lt it el pt es cs  
bg ga hr mt ro sk sl hu sr ar ja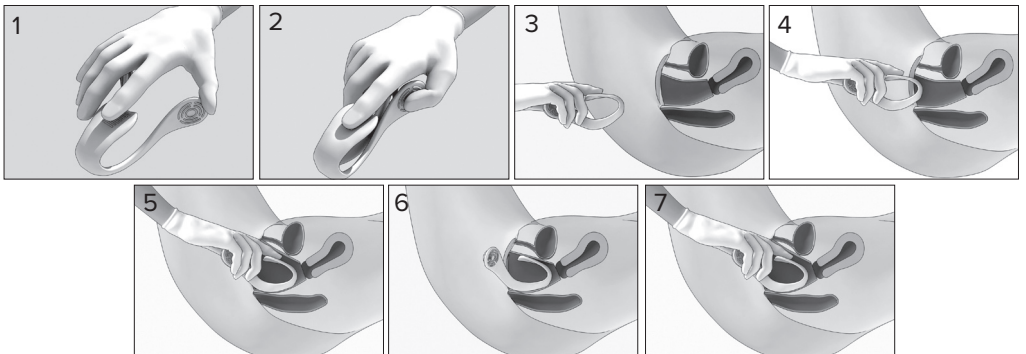

40-A1390-240214

**CE**  
0402

  
**CETRO MEDICAL**

## Hegenberger Retractor® A1390

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## Hegenberger Retractor® A1390

**Intended purpose / Indications:** The product is sterile, single use and intended to allow healthcare professionals access to postpartum inner lesions by mechanical separation of vaginal walls.

**Intended user:** The intended user is a healthcare professional, typically a midwife or obstetrician.

**Contraindication:** Not to be used in cases involving post-partum haemorrhage.

**Caution:** ⚠

- The patient must be in a supine position with elevated legs.
- Not to be used for pediatric patients or for placement in patients who have not delivered or in cases where force would be necessary for insertion.
- Do not rotate the device after vaginal insertion.
- Not to be used during pregnancy.
- Not to be used for anterior wall procedures.
- Not for reuse.

- 1. Inserting the Retractor:** Open the pouch and remove the retractor. Hold the retractor as shown in the image. Do not apply gel or lubrication.
- 2.** Gently press the sides of the retractor together, place your index finger on the top piece for control before placing in vagina.
- 3.** It is important that the patient is in a supine position with the legs elevated. The patient must not be sitting or in a semi-reclined position. This will cause the retractor to slide out.
- 4.** Keep the sides of the retractor compressed while placing the retractor until the circles on the retractor are resting behind the labia Majora.
- 5.** Ensure that the top of the retractor has been inserted past the pubic bone and ensure that the circles on the retractor are resting behind the labia Majora. Before releasing the sides, ask the patient to relax her bottom before gently releasing the sides. Do not rotate the retractor inside the patient.
- 6.** Correct placement is shown in this image. If the patient has an episiotomy or lateral tear, you rotate the retractor before placement according to where the tear is located.
- 7. Removing the Retractor:** First you gently push the retractor a little further in to disengage it from behind the pubic bone. Then you place your index finger on the top piece to ensure the retractor is clear of the cervix and the vaginal wall. Gently press the sides of the retractor together and press down on the top part and gently retract the retractor from the vagina.

All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A) Consult instructions for use B) Catalogue number C) Batch code D) Medical device E) Keep away from sunlight F) Manufacturer G) Date of manufacture H) Use-by date I) Do not use if package is damaged and consult instructions for use J) Humidity limitation 0 – 70% K) Keep dry L) Does not contain or presence of natural rubber latex M) Do not re-use N) Do not resterilize O) Temperature limit 0 - 30°C P) Sterilized using ethylene oxide Q) In accordance with the Medical Device Directive 93/42/EEC (0402). R) Caution S) Single sterile barrier system

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