

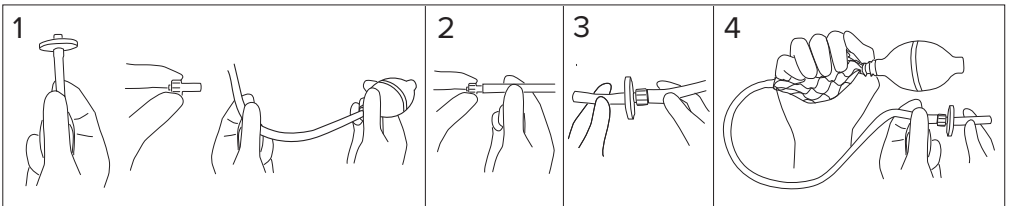
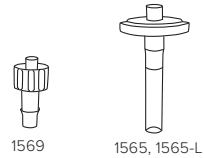
- 1562 CETRO® Bellow Reusable
- 1563 CETRO® Bellow
- 1565 CETRO® Filter
- 1565-L CETRO® Filter+
- 1566 CETRO® Pump
- 1569 CETRO® Luer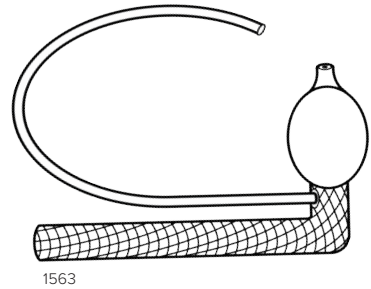

sv en no da fi de fr nl pl et lv lt it el pt es cs
 bg ga hr mt ro sk sl hu ar

40-15A0-240419

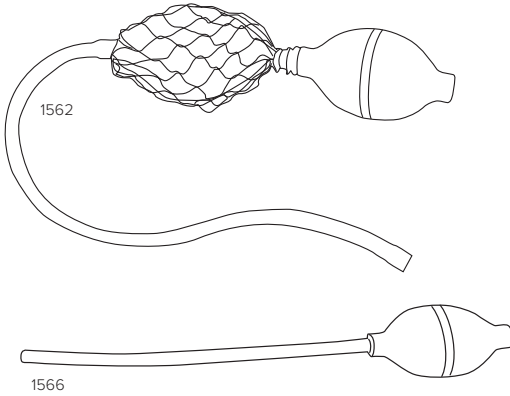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Intended Purpose: The product is an accessory to an anal endoscopic device, used by healthcare professionals during proctological examinations of rectum and anus to provide insufflation during the examination. The device is available in single use as well as in reusable versions..

Intended user: Healthcare professional, typically a proctologist or other professional working within proctology.

Caution: ⚠

- Insufflate only to dilate the tissue structures and no more than necessary to obtain a sufficient view
- Do not use any disinfectants containing alcohol. Such disinfectants may destroy the product
- Steam sterilization is not possible
- Microwave is not possible
- Alternatives to clean 1562, 1566 or 1569:
 - Clean with soap
 - Wipe-down with alcohol-free disinfection.
 - Disinfection by brief boiling: Place part into a container filled with boiling water. Make sure that all part can move about freely covered by sufficiently boiling water.
- Extended exposure to temperatures above 80°C will destroy the product. A brief temperature rise to 100°C (while boiling) can be tolerated.

☑ Applicable for article 1562 and 1566

User instruction:

1. Inspect bulb pump, filter and Luer connector, to ensure it has no damage.
2. Insert the Luer connector into the bulb pump tubing
3. Connect the filter onto the Luer connector by pressing them together.
4. Connect inflation set onto proctology equipment. Test the inflation function outside patent first.

The medical device should be disposed of as per respective hospital policy.

All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A) Consult instructions B) Catalogue number C) Batch code D) Medical device E) Keep away from sunlight F) Manufacturer G) Date of manufacture H) Use-by date I) Do not use if package is damaged and consult instructions for use J) Keep dry K) Does not contain or presence of natural rubber latex L) Do not re-use M) Contains or presence of natural rubber latex N) Not sterile O) Caution P) Complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on medical devices.

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