# WEGAA® CTG


1500-0515 WEGAA® CTG 32 mm

1500-0643 WEGAA® CTG 50 mm


9620 WEGAA® CTG 100 mm

9610-2 WEGAA® CTG 2-pack

30952090 WEGAA® CTG Button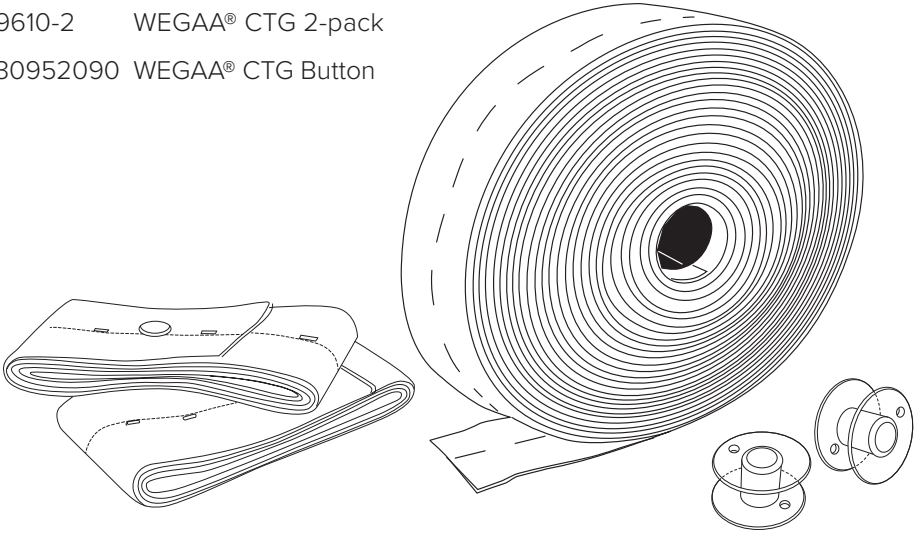

sv en no da fi de fr nl pl et lv lt it el pt es cs  
bg ga hr mt ro sk sl hu

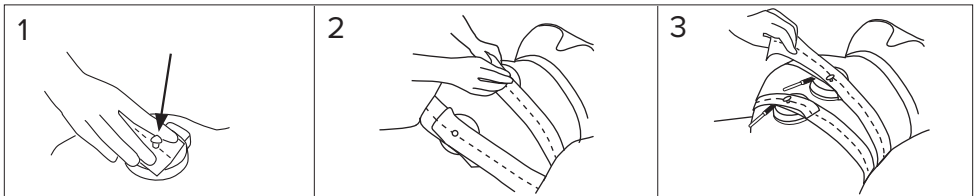40-96A0-240412

  
**CETRO MEDICAL**

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**Intended purpose:** The product is used as a belt keeping a CTG-transducer and leg plate in the correct position on the mother's body during delivery surveillance. The device is to be used by healthcare professionals and is available in single use as well as in reusable versions.

**Patient target group:** Used to fix the fetal transducer when pregnant in order to monitor the fetal heartbeat.

**Intended user:** Healthcare professionals such as midwives, gynecological nurses.

1. Attach one end of the belt to the transducer by engaging one of the holes in the belt over the button on the top of the transducer.
2. Keeping the transducer in position, tension the other end of the belt and engage the belt over the button, ensuring sufficient tension to keep the transducer in firm contact with the abdomen.
3. Avoid over-tightening as this will cause unnecessary discomfort to the patient.

The medical device should be disposed of as per respective hospital policy

All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A) Consult instructions B) Catalogue number C) Batch code D) Medical device E) Keep away from sunlight F) Manufacturer G) Use-by date H) Do not use if package is damaged and consult instructions for use I) Keep dry J) Does not contain or presence of natural rubber latex K) Do not re-use L) Washing, normal process, maximum 60 Celsius M) Non sterile N) Complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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