EasySpec® Sterile



A1211 EasySpec® XS Sterile

A1221 EasySpec® Small Sterile

A1231 EasySpec® Medium Sterile

A1241 EasySpec® Large Sterile

A1251 EasySpec® XL Sterile

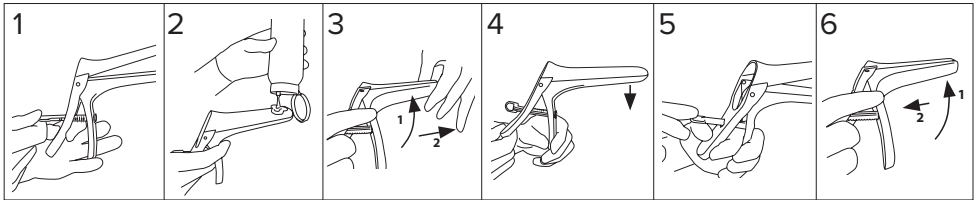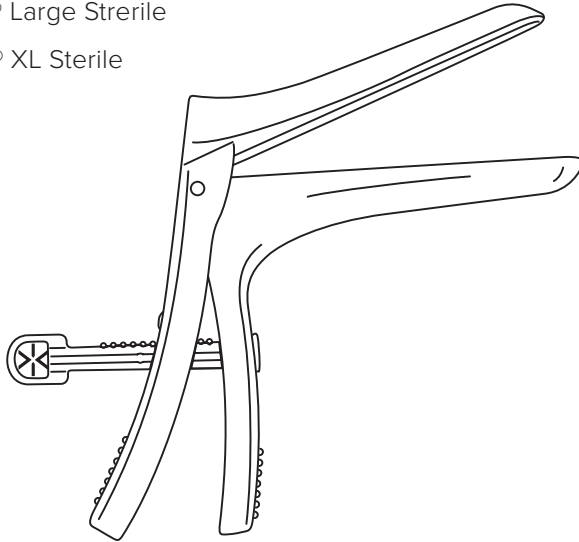sv en no da fi de fr nl pl et lv lt it el pt es cs
bg ga hr mt ro sk sl hu ar

40- A1211-A1221-A1231-A1241-A1251 -240502

CE
0402

CETRO MEDICAL

Cetro Medical AB, Nitgatan 11, SE-333 33 Smålandsstenar, Sweden, +46 371 330 30, info@cetromedical.se

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Intended purpose: The 2-part vaginal speculum is a single use, sterile device with locking mechanism to part the vaginal walls during gynaecological examinations and procedures performed by healthcare professionals.

Intended user: Healthcare professional, typically a midwife

1. Carefully remove the speculum from its packaging. Test the functioning of the speculum in locked and opened position before insertion.
2. Lock the speculum in the closed position and apply lubricating gel to the top and bottom of the beaks.
3. Hold the speculum with one hand and angle up to 90 degrees. Hold the labia apart with one hand and gently insert the speculum into the vagina.
4. Angle the speculum back to 0 degrees and gently press down against the posterior vaginal wall.
5. Loosen the lock and open the speculum so that the posterior and anterior vaginal wall are kept apart to identify the affected area. Lock the speculum in the position you want to work with.
6. At the end of the examination, loosen the lock and close the speculum and angle up to 90 degrees and carefully take it out. Be careful not to pinch the portio or vaginal mucosa when taking the speculum out.

The medical device should be disposed of as per respective hospital policy.
 All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A)Consult instructions for use B)Catalogue number C)Batch code D)Medical device E)Keep away from sunlight F)Manufacturer G)Date of manufacture H)Use-by date I)Do not use if package is damaged and consult instructions for use J)Humidity limitation 0 – 70% K)Keep dry L)Does not contain or presence of natural rubber latex M)Do not re-use N)Do not re-sterilize O)Temperature limit 0 - 30°C P)Sterilized using ethylene oxide Q)In accordance with the Medical Device Directive 93/42/EEC (0402). R)Single sterile barrier system

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