



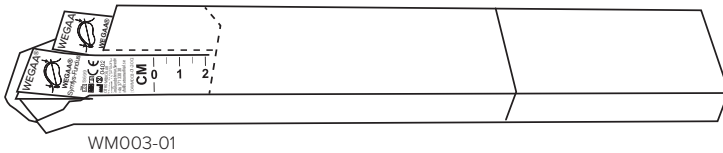
# WEGAA® Symfys-Fundus

1654 WEGAA®Symfys-Fundus Reusable

WM003-01 WEGAA® Symfys-Fundus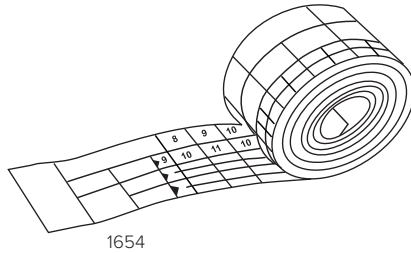

sv en no da fi de fr nl pl et lv lt it el pt es cs  
bg ga hr mt ro sk sl hu

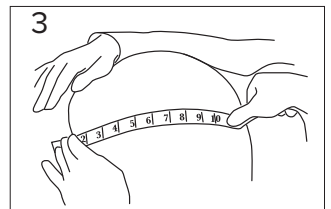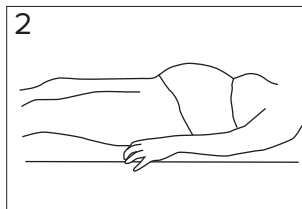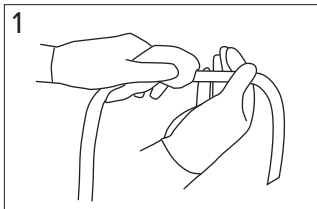40-1650-240827

CE  
0402

  
CETRO MEDICAL

## WEGAA® Symfys-Fundus 1654, WEGAA® Symfys-Fundus WM003-01

SV

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en

### Tolerance deviation ±1mm on 10 cm

**Intended purpose:** The product is a measuring tape for measurement of symphysis-fundus used by healthcare professionals. The device is available in single use as well as in reusable versions.

### Instructions:

1. (Only applicable for article "1654") Before usage, device shall be cleaned and disinfection as specified below:

**Cleaning:** The measuring tape can be washed in lukewarm water, mild soap solution without bleach.

**Disinfection:** Wipe – down the measuring tape with alcohol-based disinfectant.

2. When measuring, the patient should lie on back with outstretched legs and bladder empty.

3. Measure from the top of the symphysis to the fundus.

The medical product must be disposed of according to the respective hospital policy.

All serious incidents that occur in connection with the product must be reported to the manufacturer and competent authority.

A) Consult instructions B) Catalogue number C) Batch code D) Medical device E) Keep away from sunlight F) Manufacturer G) Date of manufacture H) Use-by date I) Do not use if package is damaged and consult instructions for use J) Keep dry K) Does not contain or presence of natural rubber latex L) Do not re-use M) Not sterile N) In accordance with the Medical Device Directive 93/42 / EEC (0402).

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