
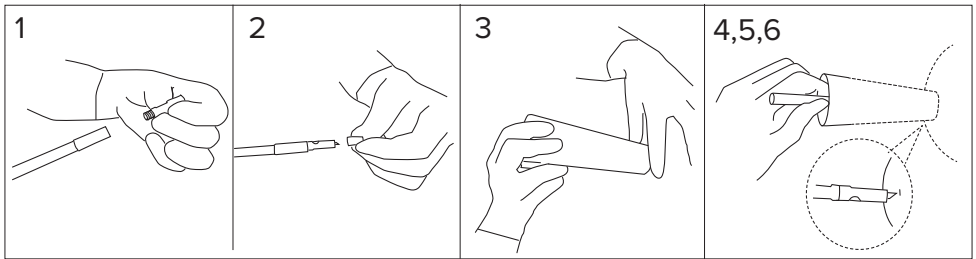
Skalpell

9602 Skalpell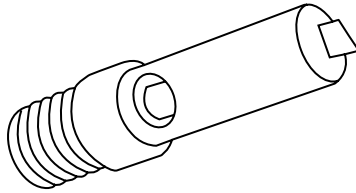

sv en no da fi de fr nl pl et lv lt it el pt es cs
bg ga hr mt ro sk sl hu

40-9602-240724

CE
0402

CETRO MEDICAL

9602 Skalpell

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9602 Skalpell

Intended purpose: The product is a single use, sterile scalpel, used by healthcare personnel in a clinical setting for blood sampling from the scalp of the fetus during childbirth.

9602 Scalpel may only be used together with the intended holder - art no. 603 - from Cetro Medical AB

User instructions:

1. Screw the scalpel onto the scalpel holder.
2. Remove the rubber cover from the scalpel blade.
3. When taking a blood sample with the 9602 Scalpel, an amnioscope must be used. Insert the amnioscope and press it firmly against the fetus's head.
4. Thoroughly clean the intended incision point with a swab
5. The incision is made by pushing the scalpel against the fetal scalp.
6. Check that the bleeding stops after the blood sample is taken.

The medical device should be disposed of as per respective hospital policy.

All serious incidents occurring in connection with the product should be reported to the manufacturer and the competent authority.

A)Consult instructions for use **B)**Catalogue number **C)**Batch code **D)**Medical device **E)**Keep away from sunlight **F)**Manufacturer **G)**Date of manufacture **H)**Use-by date **I)**Do not use if package is damaged and consult instructions for use **J)**Humidity limitation 0 – 70% **K)**Keep dry **L)**Does not contain or presence of natural rubber latex **M)**Do not re-use **N)**Do not sterilize **O)**Temperature limit 0 - 30°C **P)**Sterilized using ethylene oxide **Q)**In accordance with the Medical Device Directive 93/42/EEC (0402). **R)**Single sterile barrier system

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9602 Skalpel smete uporabljati samo z enim nameranimi skrbnikom- izd. št. 603 - iz Cetro Medical AB
A 9602 Skalpell szike csak rendeltetésszerű tartójával használható - cikkszám 603 - gyártja a Cetro Medical AB